

Stellar M22

Shine on

Stellar™ IPL

One Applicator. A Galaxy of Treatments.




9 Filters and Lightguides, changeable in seconds, to treat over 20 skin conditions.



The Stellar OPT® advantages

- ◆ Control pulse shape by equalizing energy distribution over the entire pulse or sequential pulses
- ◆ Gentler, more comfortable, patient-friendly procedures, with lower effective fluences
- ◆ High peak power, shorter pulses – optimal for IPL skin treatments using photorejuvenation and treatment of benign pigmented lesions
- ◆ Advanced OPT will also allow determining the specific fluence per sub-pulse when using multiple-sequential pulsing (MSP™), for fine-tuned treatment settings.

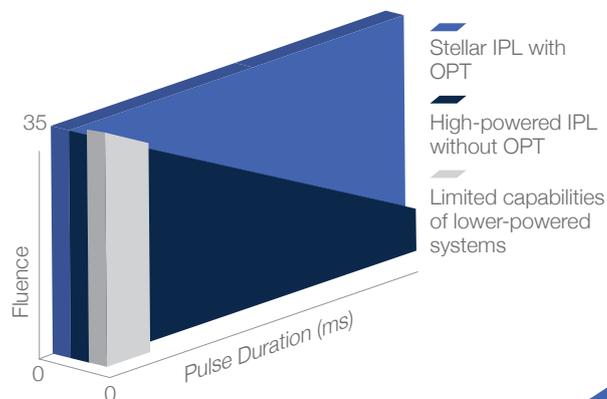

Treat more than 20 skin conditions

Including IPL photorejuvenation treatments, vascular lesions, pigmented lesions, active acne and hair removal

Typical Clinical Results



Courtesy Daniel Friedmann, MD



Courtesy Diana Demidion, MD




Courtesy of Diana Demidion, MD



Courtesy of Bill Che-Hao Hsu, MD.



Courtesy of Mariela Nazar, MD.

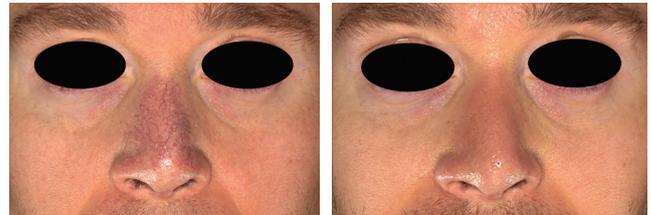

Courtesy of Matteo Tretti Clementoni MD.

Warnings and risks: Stellar™ IPL is contraindicated for patients with active infection, viral, fungal or bacterial diseases. The use of Stellar IPL could cause redness, swelling, scarring, change of pigmentation. See the system user manual for a complete list of contraindications and risks.

Stellar™ IPL Module Specifications

Spectrum	400-1200 nm
Expert Filters	515, 560, 590, 615, 640, 695, 755 nm, Acne (400-600 & 800-1200 nm), Vascular (530-650 & 900-1200 nm)
Fluence	10-56 J/cm ²

Pulse Delay	5-150 ms
Pulse Duration	4-20 ms
Pulse Characteristic	Multiple Sequential Pulsing
Repetition Rate	Up to 1 Hz
Spot Size	35 x 15 mm ² ; 15 x 8 mm ² , 6 mm
Cooling	Continuous contact cooling

Stellar M22, Stellar IPL, OPT, MSP, ExpertFilters, SapphireCool lightguides are trademarks or registered trademarks of the Lumenis Group of Companies. Specifications are subject to change without notice Lumenis is ISO 13485:2016 certified.

Lumenis Ltd.
Yokneam Industrial Park
6 Hakidma Street P.O.B. 240
Yokneam 2069204, Israel
Tel: +972-4-959-9000

EMEA
Dreieich Dreieichenhain,
Germany
T +49 6103 8335 0
F +49 6103 8335 300

JAPAN
Tokyo, Japan
T +81 3 6743 8300
F +81 3 6743 8301

ASIA PACIFIC
Gurgaon, India
T +91 124 422 07 95

Kowloon, Hong Kong
T +852 217 428 00
F +852 272 251 51

Sydney, Australia
T +61 1800 5863 64

Roma, Italy
T +39 06 90 75 230
F +39 06 90 75 269

AMERICAS
San Jose, CA, USA
T +1 408 764 3000
+1 877 586 3647
F +1 408 764 3999

Lumenis (Germany) GmbH
Heinrich-Hertz-Str 3 D-63303
Dreieich-Dreieichenhain
Germany
Tel: +49 (0) 6103 8335 0



Rx ONLY

© 2020 LUMENIS LTD., ALL RIGHTS RESERVED

