



# Stellar M22

Shine on



## Q-Switched Nd:YAG

7 spot sizes for customized treatments, 4 of which are available also as disposable tips



2mm



2.5mm



3.5mm



5mm



2mm



2.5mm



3.5mm



4mm



5mm




6mm




8mm

### “Top-Hat” beam profile advantages

- ◆ Enhanced treatment safety due to avoidance of bulk heating using the homogenizer
- ◆ Top-hat beam profile minimizes epidermal damage, tissue textural changes and the occurrence of scarring
- ◆ A lack of an homogenizer might result in hot spots within the beam and increased risk of scarring

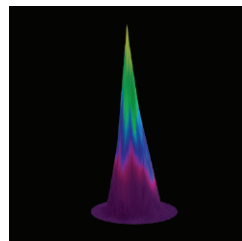


Beam profile side view without an homogenizer

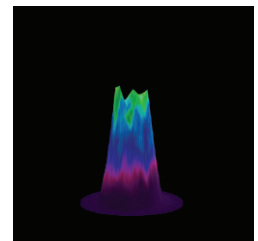


Beam profile side view with an homogenizer for a top-hat beam

# For skin toning by treatment of pigmented lesions, dark tattoo removal and advanced sequential treatments with other Stellar M22 modules


## Typical Clinical Results

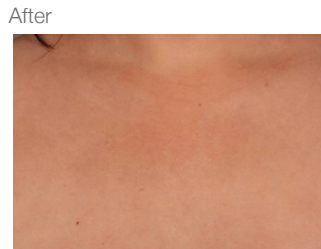

Courtesy of Panta Rouhani Schaffer, MD, PHD, MPH

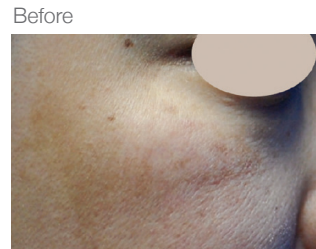



Courtesy of Yuanhong Li, MD.

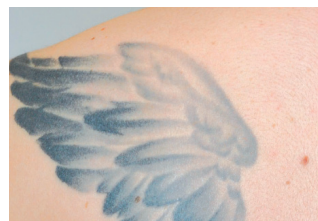


Courtesy of Michael Oeser, MD.



Courtesy of Moshe Lapidot, MD.

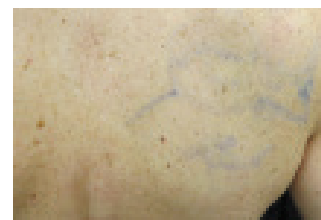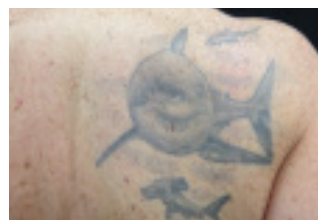

Lumenis' in-house clinic

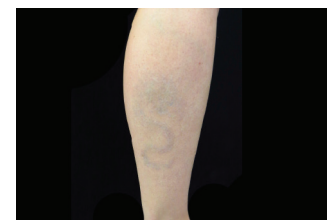

Lumenis' in-house clinic

**Warnings and risks:** The Stellar M22™ Q-Switched Nd:YAG is contraindicated for patients with skin disorders or inflammatory skin conditions. The use of the Q-switch could cause redness, swelling, pigmentation changes. See the system user manual for a complete list of contraindications and risks.

### Q-Switched Nd:YAG module specifications

Wavelength	1064 nm	Repetition Rate	0.5-5.0 Hz
Fluence	0.9-14 J/cm <sup>2</sup>	Spot Size	2, 2.5, 3.5, 4, 5, 6 and 8 mm
Pulse Duration	6-8 ns		

The Lumenis Group of Companies, Lumenis, its logo, Stellar M22 are trademarks or registered trademarks of the Lumenis Group of Companies. Specifications are subject to change without notice. Lumenis is ISO 13485:2016 certified.

**Lumenis Ltd.**  
Yokneam Industrial Park  
6 Hakidma Street P.O.B. 240  
Yokneam 2069204, Israel  
Tel: +972-4-959-9000

**Lumenis (Germany) GmbH**  
Heinrich-Hertz-Str 3 D-63303  
Dreieich-Dreieichenhain  
Germany  
Tel: +49 (0) 6103 8335 0

**EMEA**  
Dreieich Dreieichenhain,  
Germany  
T +49 6103 8335 0  
F +49 6103 8335 300

**Roma, Italy**  
T +39 06 90 75 230  
F +39 06 90 75 269  
**Hertfordshire, UK**  
T +44 20 8736 4110  
F +44 20 8736 4119

**JAPAN**  
Tokyo, Japan  
T +81 3 6743 8300  
F +81 3 6743 8301

**AMERICAS**  
San Jose, CA, USA  
T +1 408 764 3000  
+1 877 586 3647  
F +1 408 764 3999

**ASIA PACIFIC**  
Gurgaon, India  
T +91 124 422 07 95  
Kowloon, Hong Kong  
T +852 217 428 00  
F +852 272 251 51

Sydney, Australia  
T +61 1800 5863 64



Rx ONLY

© 2020 LUMENIS LTD., ALL RIGHTS RESERVED



LUMENIS.COM/Stellar-M22

PB-00002490 Rev A